PL**Alkomat
Instrukcja obsługi****EN****Alcohol tester
User's manual****CS****Alkohol tester
Návod k obsluze****SK****Alkohol tester
Návod na použitie****HU****Légmérő
Használati útmutató****BG****Алкотестер
Ръководство за употреба****LT****Alkotesteris
Naudojimo instrukcija****LV****Alkometrs
Lietošanas instrukcija****ET****Alkomeeter
Kasutusjuhend****RO****Alcomet
Manual de utilizare**

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Thank you for purchasing the breathalyzer. Please read this user manual carefully. Doing so will allow you to utilize the full capabilities of the device and use it correctly. Please also keep this manual in an accessible place in case it is needed in the future.

Before turning on the product for the first time, please carefully read the following points:

1. The device should not be considered as a decisive factor in determining whether to drive after consuming alcohol.
2. The test results provide a general indication and should be treated as an overall guideline. Even with the same alcohol consumption and blood alcohol concentration, intoxication levels may vary depending on individual physical condition and differences among test subjects.
3. Regardless of whether this breathalyzer is used for testing, neither the manufacturer nor the seller is responsible for driving under the influence or any other legally prohibited actions.
4. Alcohol absorption into the bloodstream takes approximately 20 minutes. Therefore, it is recommended to perform the test after about 20 minutes. If the test is conducted immediately after alcohol consumption, the results may not accurately reflect the blood alcohol concentration.
5. To obtain more accurate readings, the interval between measurements should be longer than 3 minutes.
6. Do not disassemble or wash the device.
7. When measuring with low battery power, there may be a discrepancy between the test result and the actual value. Ensure that AAA batteries are adequately charged.

8. Store the device away from direct sunlight and heating appliances.

DESCRIPTION OF THE DEVICE

- ① Air intake
- ② Indicator light
- ③ LCD screen
- ④ Power button
- ⑤ Speaker
- ⑥ LED flashlight
- ⑦ Air outlet

PREPARATION BEFORE FIRST USE

1. Prepare 3 x 1.5V (AAA) batteries, remove the battery cover, insert the batteries, and replace the cover. Batteries are not included!
2. Press the Power button.
3. The device will warm up for approximately 0-10 seconds, followed by a 10-second countdown—wait until the heating process is complete.
4. After warming up, the device will automatically start a 6-second air-blowing countdown. After exhaling steadily for the required duration, wait for the test results to appear.
5. After completing steps 2, 3, and 4, the result will appear on the display within 5 seconds.
6. Setting the measurement unit: Press once to switch between different measurement units (% BAC / mg/L BrAC).
7. To turn on the emergency LED light, press and hold the Power button. If the breathalyzer is used for the first time after being taken out of the packaging or left unused for

a long period, gases and other contaminants may have accumulated on the sensor. In such a case, press the Power button several times to clear the breathalyzer by preheating it.

PREPARATION FOR OPERATION

1. Press and hold the Power button for about 2 seconds. A „beep” sound will be heard, and the device's display will turn on.
2. The device will enter the warm-up mode. The screen will count down from 10 to 0, after which the breathalyzer will be ready for testing.

TESTING PROCEDURE

1. After warming up, proceed to the breath test phase. The screen will show a 6-second countdown. Blow steadily into the installed mouthpiece for about 5 seconds until a sound signal is heard. Shortly after, the test result will be displayed.
2. The numeric result will also be accompanied by an appropriate LED light indicator.

Green light – Within acceptable limits

If the test result is $< 0.3\text{g/L}$, the green light turns on.

If the test result is $< 0.030\%$ BAC, the green light turns on.

If the test result is $< 0.15\text{mg/L}$, the green light turns on.

Yellow light and alarm sound – Alcohol consumption detected

Test result falls within $\geq 0.3\text{g/L}$ to $< 0.8\text{g/L}$.

Test result falls within $\geq 0.030\%$ BAC to $< 0.080\%$ BAC.

Test result falls within $\geq 0.15\text{mg/L}$ to $< 0.25\text{mg/L}$.

Red light and alarm sound – Intoxicated state

Test result exceeds $\geq 0.8\text{g/L}$.

Test result exceeds $\geq 0.080\%$ BAC.

Test result exceeds $> 0.25\text{mg/L}$.

MEASUREMENT UNITS

Different units used in various countries:

- 1mg/100mL – China
- % BAC – USA
- g/L – Europe
- mg/L BrAC – Japan

0.05% BAC means 0.05g (50mg) of alcohol per 100mL of blood.

BAC (Blood Alcohol Concentration) – Alcohol level in the blood					BrAC (Breath Alcohol Concentration) – Alcohol level in exhaled air	
	mg/100ml	%	‰	g/l	mg/l	PPM
1	10	0.01	0.1	0.1	0.05	25
2	20	0.02	0.2	0.2	0.10	50
3	30	0.03	0.3	0.3	0.15	75
4	40	0.04	0.4	0.4	0.20	100
5	50	0.05	0.5	0.5	0.25	125
6	60	0.06	0.6	0.6	0.30	150
7	70	0.07	0.7	0.7	0.35	175
8	80	0.08	0.8	0.8	0.40	200
9	90	0.09	0.9	0.9	0.45	225
10	100	0.10	1.0	1.0	0.50	250
11	110	0.11	1.1	1.1	0.55	275
12	120	0.12	1.2	1.2	0.60	300
13	130	0.13	1.3	1.3	0.65	325
14	140	0.14	1.4	1.4	0.70	350
15	150	0.15	1.5	1.5	0.75	375
16	160	0.16	1.6	1.6	0.80	400
17	170	0.17	1.7	1.7	0.85	425
18	180	0.18	1.8	1.8	0.90	450
19	190	0.19	1.9	1.9	0.95	475
20	200	0.20	2.0	2.0	1.00	500

SUSPECTING A MALFUNCTION?

Before requesting a replacement or return, check the following:

1. Is a strong odor coming from the sensor?
 - Press the Power button and blow air more than 3 times to clean the sensor.
2. Was the test conducted in a location with noticeable odors?
 - Change location before testing.
3. Was the test conducted immediately after a meal?
 - Perform the test again 20 minutes after eating.
4. The test was not completed:
 - Was the exhalation too weak? Try exhaling more forcefully and steadily.
 - Was the mouthpiece too far from the mouth? Position the device correctly.
 - Was the test conducted in a windy location? Move to a different area before testing.
5. Undefined error:
 - Was the test conducted in a windy location? Move to a different area before testing.
 - Was the test conducted in an area with strong odors? Change location before testing.
6. I haven't been drinking, but the result shows alcohol:
 - Alcohol residues may accumulate inside the alcohol sensor and air intake. Press the Power button and blow air more than 3 times to clean the sensor.
 - Fermentation gases produced in the body can react with the sensor.
7. Pressing the Power button does not turn on the display:
 - Is the battery low? Try replacing the batteries with new ones.

Power consumption:	120 mA 4.5V
Battery:	Batteries: 3 x AAA 1.5V (not included)
Temperature range:	-20°C~50°C
Sensor type:	Semiconductor sensor
Alcohol concentration measurement range:	0-200mg/100mL
Product dimensions:	90mm × 47mm × 23mm
Product weight:	45g

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The symbol of a crossed-out wheeled bin placed on electronic or electrical equipment, its packaging or accompanying documents means that the product may not be thrown out together with other waste. Used equipment may contain substances with toxic and carcinogenic properties, hazardous to human health and life, and poisoning the soil and groundwater. It is the user's responsibility to hand over the used equipment to a designated collection point for its proper processing. For more information on recycling of electronic and electrical equipment, please contact your local authorities, waste disposal services and the place where you purchased this product.

The household plays an important role in contributing to the reuse and recovery of secondary raw materials, including recycling, waste equipment. At this stage, attitudes are formed that affect the preservation of the common good, which is a clean natural environment.

tracer[®]

Producent:

Megabajt Sp. z o.o., ul. Rydygiera 8, 01-793 Warszawa
e-mail: info@megabajt.com.pl, tel. +48 22 560 73 00